


RAZ shower chair toilet compatibility

Standard WC- Adjusting the height of your RAZ for clearance

The table below shows the seat height setting you will need to clear the listed toilet bowl heights. It is important to make a note of the toilet bowl height when carrying out a client assessment and submitting this information when getting a quotation or ordering from core stock.


Toilet bowl height (mm)	360	380	400	420	440	460	480	500	520	540	560
ART	500	530	530	560	560	580	610	640	640	660	685
AT standard seat height		530	530	560	560	580	610	640	640		
ATP standard seat height		530	530	560	560	580	610	640	640		
AP/SP standard seat height		500	520	540	560	580	600				
AT600 5" castor seat height		570	570	570	570	570	600				
AP600 5" castor seat height		490	510	540	570	570	600				
SP600 5" castor seat height		490	510	540	570	570	600				

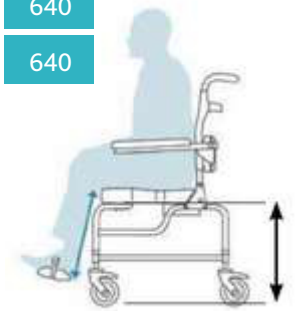

Measurement from floor to bottom of seat

Lining up your RAZ with the toilet bowl

Getting this measurement correct is important so that the seat aperture lines up with the toilet opening. Remember to consider waste pipe location and the toilets distance from the wall.

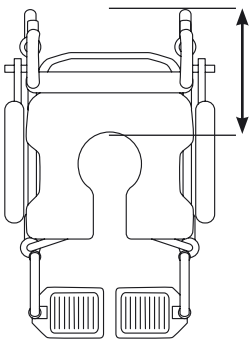

Rearmost chair point to rear of seat aperture	
ART	230
AT (attendant tilt)	290
ATW (attendant tilt wide)	290
ATP (attendant tilt paediatric)	230
AP (attendant propel)	180
AP Auto (attendant propel wide)	180
SP (self propel)	280
AT600 (attendant tilt bariatric)	350
AP600 (attendant propel bariatric)	255
SP600 (self propel bariatric)	380

BEST PRACTICE

A common requirement is to have the skirt accessory added to the underside of the seat and the option of removing the pan sliders to provide a closer fit with the toilet. The standard moulded seat shaping contributes to a much improved wash experience.



Bio Bidet - The RAZ range is not compatible with a Bio Bidet.

RAZ shower chair toilet compatibility

Auto WC Compatibility

See below which RAZ chairs are compatible with popular automatic toilets.

If your clients automatic toilet is not listed below then please contact our sales team to discuss.

ART



- ✓ Geberit Mera
- ✓ Geberit 7000
- ✓ Aquaclean 8000
- ✓ Closomat Samoa / Palma Vita

AT



- ✓ Geberit Mera

ATW



- ✓ Geberit Mera
- ✓ Geberit 7000
- ✓ Aquaclean 8000
- ✓ Closomat Samoa / Palma Vita

AP & SP



- ✓ Geberit Mera

AP GEB



- ✓ Geberit Mera
- ✓ Geberit 7000
- ✓ Aquaclean 8000
- ✓ Closomat Samoa / Palma Vita

AT/AP/SP 600



- ✓ Geberit Mera

RAZATP



- ✓ Geberit Mera

Assessment Considerations

- The size and shape of the room
- Turning circle
- Obstructions around the toilet- grab handles, basins etc
- Soil stack location and direction
- Flooring and thresholds
- Remember the RAZ will need to be in an upright position when over a WC.

