

## Manually Operated Chair Guide

### Tilt-In-Space

- 1** Pull the lever to release the tilt



- 2** Carefully tilt the chair back



#### TO RETURN TO STANDARD POSITION

Pull the lever to release the tilt, and carefully tilt the chair back up.

### Leg Rest

- 1** Pull the release lever



- 2** Pull up the leg rest



#### TO CLOSE THE LEG REST

Pull the release lever and push the foot rest down to the closed position.

**0845 658 8411**   [sales@wealdenrehab.com](mailto:sales@wealdenrehab.com)

[www.wealdenrehab.com](http://www.wealdenrehab.com)


  [wealdenrehab](https://www.facebook.com/wealdenrehab)



**Wealden Rehab**  
Care Equipment Excellence



# Electrically Operated Chair Guide

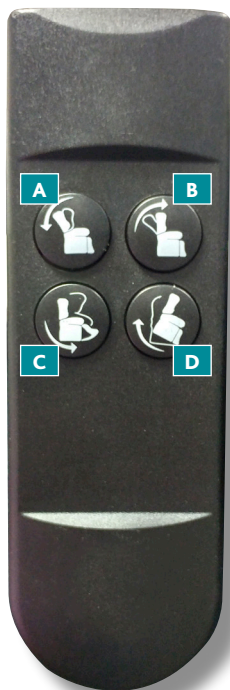

## Tilt-In-Space

Press **A** to Tilt

Press **B** to Return



## Leg Rest

Press **C** to Elevate

Press **D** to return



0845 658 8411 [sales@wealdenrehab.com](mailto:sales@wealdenrehab.com)

[www.wealdenrehab.com](http://www.wealdenrehab.com)

  [wealdenrehab](https://www.facebook.com/wealdenrehab)



**Wealden Rehab**  
Care Equipment Excellence